



# Compatibility of Products with Site-Care Agents

This chart indicates the compatibility of our products with commonly used site care formulations. It is neither an all inclusive nor exclusionary listing. Please contact the Compliance Department with any specific questions concerning site care compatibility.

CATHETER NAME	MATERIAL	CHLORHEXIDINE GLUCONATE 2% & 4%	BETADINE® SOLUTION (10% POVIDONE IODINE)	70/30% ALCOHOL	HYDROGEN PEROXIDE	<0.057% SODIUM HYPOCHLORITE	ANTIMICROBIAL OINTMENTS & CREAMS (MUPIROCIN, POLYMYXIN)	SILVER SULFADIAZENE CREAM 1%
DUO-FLOW® (400 XL)	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
SOFT-LINE®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
DUO-SPLIT®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
TRI-FLOW™	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
FEMORAL	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
SUBCLAVIAN	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
HEMO-CATH®	Silicone	Yes	<b>NO</b>	Yes	<b>NO</b>	<b>NO</b>	Yes	Yes
FREE-FLOW®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
SPLIT CATH®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
SLX	Silicone	Yes	<b>NO</b>	Yes	<b>NO</b>	<b>NO</b>	Yes	Yes
T3	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
TRIO-CT™	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
DUO-FLOW® SXS	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
DUO-FLOW® III	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
BIOFLEX® TESIO®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
HEMO-FLOW®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
SPLIT STREAM®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
TITAN HD™	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
ESCHELON™	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
DUO-FLOW®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
SYMETREX®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
PERITONEAL	Silicone	Yes	<b>NO</b>	Yes	<b>NO</b>	<b>NO</b>	Yes	Yes
MULTI-CATH	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
LT CVC	Silicone	Yes	<b>NO</b>	Yes	<b>NO</b>	<b>NO</b>	Yes	Yes
VASCU-PICC®	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
PRO-PICC® CT	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes
PRO-LINE® CVC	Polyurethane	Yes	Yes	Yes	Yes	Yes	Yes	Yes